


MAGRINA-BOOKWALTER® VAGINAL RETRACTOR KIT

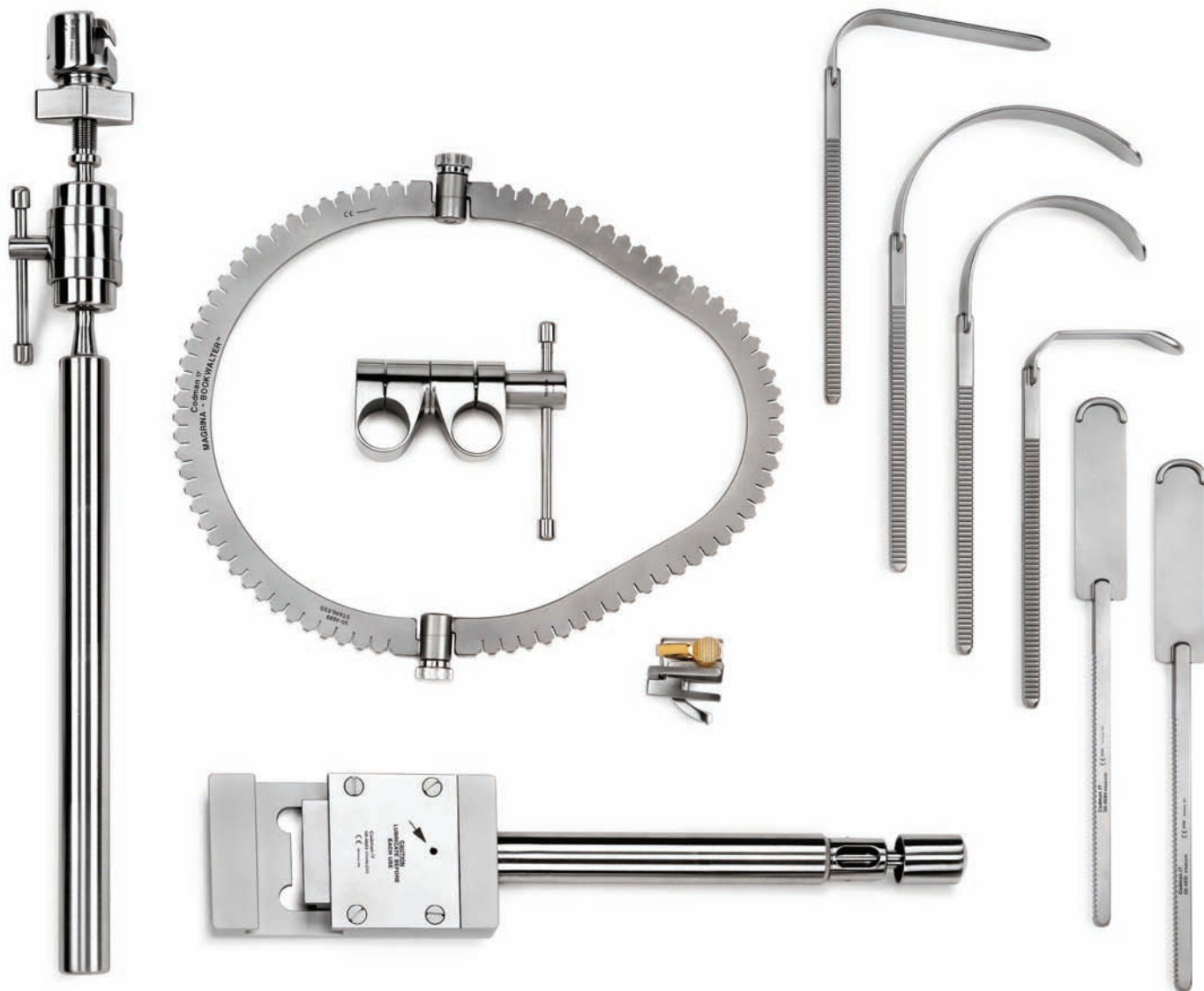

New Kit configurations provide options for optional exposure.

Codman
a Johnson & Johnson company

Magrina-BOOKWALTER Vaginal Retractor Kit


- **Single Post Fixation** attaches easily to the table making set-up quick and easy while maintaining sterile technique.
- **Horizontal Flex Bar** adapts easily to different size patients.
- **The Ratchet Mechanism** affords easy adjustment of retractor tension and depth as required during the procedure.
- **Two Deaver Retractor Blade Sizes.** Small and large Deaver retractor blades afford optimal exposure before and after entering the anterior cul-de-sac for retraction of the bladder.
- **Vaginal Ring Angulation.** 20-degree angulation contours to the patient's anatomy, affording retraction from two separate planes.



With the end of the O.R. table dropped, table post parallel to the floor, and the patient in the high-lithotomy position, the vaginal ring is positioned with the 20-degree break at the level of the posterior vaginal wall.



The distal end of the horizontal flex bar is connected to the table post and tightened via the post coupling. Grasp the vaginal ring at the 5 o'clock position with the horizontal flex bar, and secure.

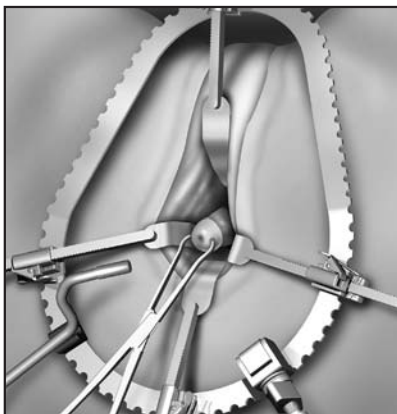

With posterior (6 o'clock), lateral (4 and 8 o'clock) and Deaver (12 o'clock) retractor blades positioned, optimal exposure is realized.

Magrina-BOOKWALTER Segmented Pelvic Retractor Kit (50-4692)

Code	Description
50-4689	Magrina-Bookwalter Segmented Pevlic Ring
50-4591	Blade Assembly, Lateral, Vaginal Retractor
50-4592	Blade Assembly, Deaver, Small Vaginal Retractor
50-4593	Blade Assembly, Deaver, Large Vaginal Retractor
50-4594	Blade Assembly, Posterior, Vaginal Retractor
50-4691	Bookwalter Super Malleable Blade with Lip, 1x4 (2 ea.)
50-4694	Bookwalter Malleable Blade with Lip, 1x4 (2 ea.)
50-4554	Bookwalter Post Coupling, 1" x 1"
50-4580	Bookwalter Titl Ratchet Mechanism (6 ea.)
50-4582	Bookwalter Horizontal Flex Bar
50-4683	Bookwalter Table Post

Magrina-BOOKWALTER Pelvic Retractor Kit (50-4693)

Code	Description
50-4690	Magrina-Bookwalter Pevlic Ring
50-4591	Blade Assembly, Lateral, Vaginal Retractor
50-4592	Blade Assembly, Deaver, Small Vaginal Retractor
50-4593	Blade Assembly, Deaver, Large Vaginal Retractor
50-4594	Blade Assembly, Posterior, Vaginal Retractor
50-4691	Bookwalter Super Malleable Blade with Lip, 1x4 (2 ea.)
50-4694	Bookwalter Malleable Blade with Lip, 1x4 (2 ea.)
50-4554	Bookwalter Post Coupling, 1" x 1"
50-4580	Bookwalter Titl Ratchet Mechanism (6 ea.)
50-4582	Bookwalter Horizontal Flex Bar
50-4683	Bookwalter Table Post

Magrina-BOOKWALTER Vaginal Retractor Kit (50-4547)

Code	Description
50-4590	Magrina-Bookwalter Vaginal Retractor Ring I.D. 8" x 10.5" (203mm x 206mm)
50-4591	Blade Assembly, Lateral, Vaginal Retractor (2 ea.)
50-4592	Blade Assembly, Deaver, Small Vaginal Retractor
50-4593	Blade Assembly, Deaver, Large Vaginal Retractor
50-4594	Blade Assembly, Posterior, Vaginal Retractor
50-4554	Bookwalter Post Coupling, 1" x 1"
50-4582	Bookwalter Horizontal Flex Bar
50-4681	Bookwalter Table Post, 1" Diameter
50-4580	Bookwalter Titl Ratchet Mechanism (4 ea.)